

Sieco Tape



Sieco of France, a specialist in healthcare products, has launched **Siecollodion**, a hypoallergenic latex-coated cotton gauze that sticks only to itself. It stops bleeding (haemostatic) and helps to heal wounds whilst still allowing the wound to breathe. It is resistant to water, alcohol and household detergents, so it enables users to take a bath or shower without getting the wound wet or displacing the bandage. Ordinary cotton or self-adhesive plasters that allow the skin to breathe are often waterproof, which can delay or complicate the healing process.

Sieco also manufactures **Siecopeau**, a fully waterproof haemostatic latex bandage which sticks only to itself. It is applied to create a perfectly fitted and sealed protection around fingers, and as it is elastic it conforms to finger shape - allowing users to continue with normal activities. It is an excellent wound dressing, particularly in wet areas, and can be applied immediately over antiseptic solutions. With gentle pressure, it stops and contains blood flow. It is also recommended for use alone on fingers to prevent cuts, scratches or blisters.

The **Sieco Personal Bandage Kit** goes where you go. It is a convenient kit containing

- one roll of **Siecollodion**,
 - one roll of **Siecopeau**
 - a pair of quality stainless steel safety scissors
- in a waterproof plastic container.


This great value product is ideal for tradespeople, campers, gardeners, sportspeople, fishermen, caterers, butchers, travellers, children, the elderly and virtually anyone. It's small enough to go anywhere with you - in the glovebox of your car, caravan or boat, or throw it in your tackle box or toolkit.

- **Fast Application: For wet or dry wounds on fingers and toes**
- **Dexterous: Conforms to finger shape, allowing users to continue with normal activities**
- **Non-Stick: Sieco has no adhesive. It won't stick to skin, wounds or hair and is ideal for frail or elderly skin.**
- **Water Resistant: Siecollodion repels liquids and dirt, yet allows the wound to breathe**
- **Waterproof: Siecopeau is a fully waterproof protection that acts like a second skin.**
- **TGA Approved Class 1 Medical Device**
- **C/E Medical Certificate Category**
- **Over 15 Million rolls sold throughout Europe**

Siecollodian

Instructions:


1. Unroll and tear a sufficient length of binding
2. Apply binding on the finger, the paper sheet outside.
3. Squeeze together the two ends of binding, withdraw the paper sheet
4. Adjust the bandage to marry the shape of the finger.
5. Fold back the surplus of binding
6. Use scissors to remove.

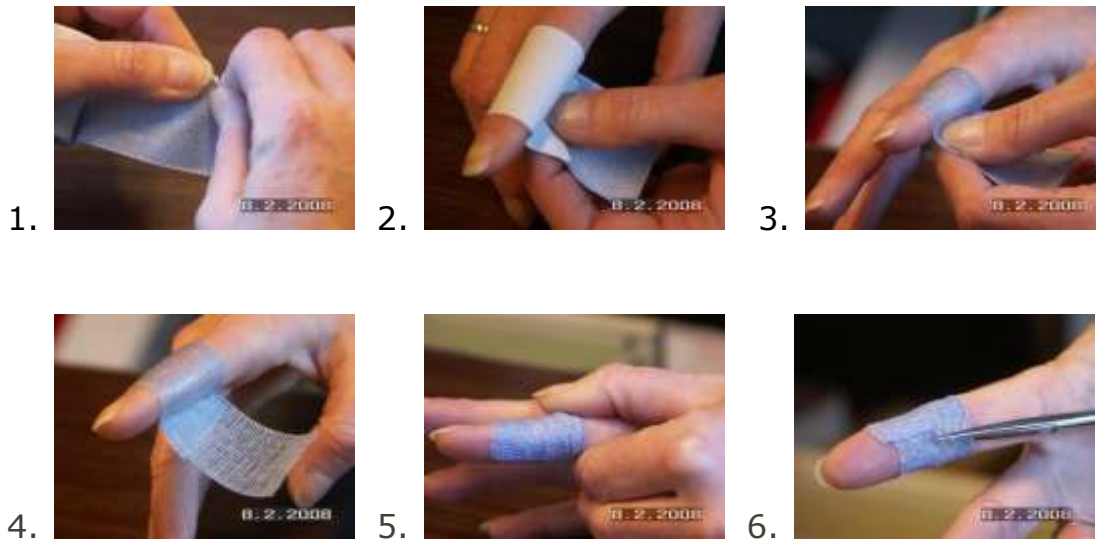

Siecopeau

Instructions:

1. Unroll and cut a sufficient length of binding
2. Remove protective film
3. Apply directly to the skin
4. Wrap around the finger or toe and squeeze together the two ends of binding
5. Fold back the surplus of binding, or trim with scissors
6. Shape to required form

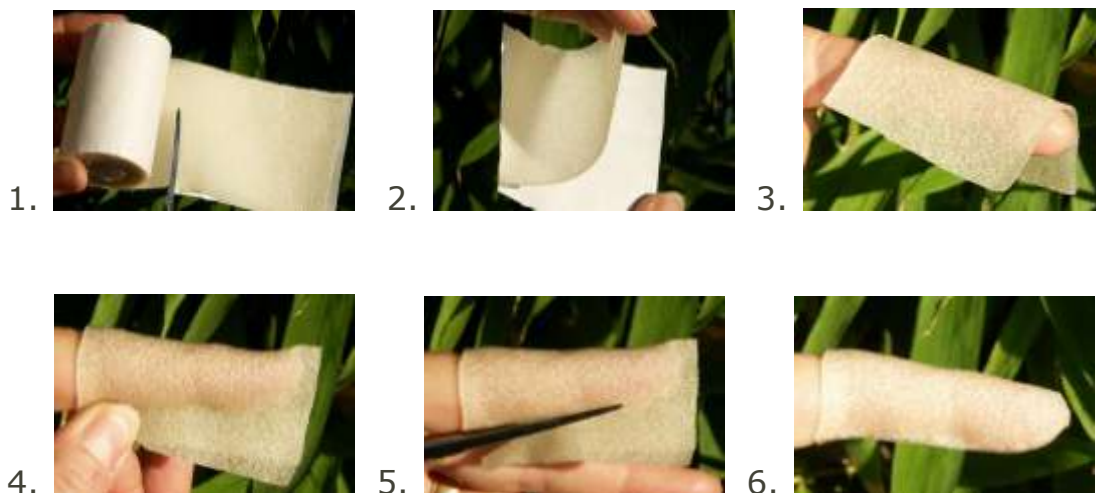

Testimonials

Blisters seem to happen to me whenever I pick up a rake or shovel. I don't know why. Doh! Anyway, when I am in the garden and I get a blister I now cover it with the Siecopeau bandage and just keep on working. It's the best thing to do because getting started is the hardest thing to do and when I am on a role I don't want to stop just because of a blister. I have a mate that is a "brickie" and he uses the Siecopeau when the skin on his fingers splits and cracks in the cold weather. It's better than using electrical tape.

Bill, Osborne Park WA 2009

I used Siecollodion on my finger after I sliced my knuckle while grating cheese. I doubled it over to stop the bleeding and it stayed on even though my hands were wet. I recommend it.

Laura, Currambine WA 2008

John gave me some Sieco gauze bandage to keep on my boat and to use for evaluation. Well it only took 2 days before I needed to use it, but not on the boat. I burnt my finger on the barbeque and it blistered quite badly. I covered it with a plaster I had at home and forgot about the Sieco gauze on the boat. Unfortunately the burn was stinging a lot so I removed the plaster and could easily see it wasn't helping at all. The skin around the wound had gone completely white (it looked dead) and the wound was raw. Then I remembered the gauze and that John had said it would help a wound to breathe and protect it at the same time so I applied it as I was shown. The difference was amazing and the burn healed almost completely in just a few days. Now I carry the Sieco gauze in my glovebox and on the boat as well as in the workshop at home. It really works

Brian, Malaga WA 2008

This April on a six day sea kayak expedition to the islands off Exmouth I was thrilled to be able to use your Sieco Gauze and Latex products. As I always suffer from callouses and blisters, by the end of an expedition I usually have extremely painful hands and use rolls of the old sticky plaster tapes. Although these work they have a couple of problems. 1) The tape has to be applied when your hands are absolutely dry. This is not necessary with the Sieco bandages because they can be applied whilst the hands are still moist. 2) The Sieco tape can be easily reapplied after stopping. The other plasters tend to have to keep on all day which means they begin to disintegrate and usually leave adhesive bits on the paddle and fingers which become annoying and they are not as effective as Sieco Tape.

Wendy, Ocean Reef WA 2009

On the same expedition with Wendy and whilst fishing I managed to acquire a neat line cut, which was worth it considering the size of the fish I caught (a spanish mackerel) and the Sieco gauze tape worked wonders because I managed to apply it whilst my hands were still wet! It's a great product for us outdoors types and I wished I had it all the years I have been kayaking.

Tel, Ocean Reef WA 2009